Simbologia | Simbología | Symbology

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POWER BLEACHING 37% Office

PRESENTATIONS:

- 3 syringes with 3 g each and 3 tips.
- 1 syringe with 3 g and 1 tip.

DESCRIPTION:

The Power Bleaching system consists of a 37% carbamide peroxide gel exclusively intended for professional in-office whitening. It is used for the whitening of vital and non-vital teeth with scientific technology to provide the desired result with maximum comfort during treatment. The whitening effect of the product can already be observed in the first treatment sessions. Its formulation and use protocol were developed to provide reduced sensitivity resulting from the treatment, reduced application time, adaptation to the patient's routine and easy handling, all this with the objective of recovering the aesthetics of the teeth through excellent whitening.

COMPOSITION:

Carbamide Peroxide, Potassium Oxalate, Sodium Fluoride, Thickener, Neutralizer, Humectant, Purified Water.

INDICATION:

It is intended for tooth whitening in the following cases: **Vital Teeth:** naturally dark or yellowish, darkened by dyes, by age, by trauma, stained to a mild degree by tetracycline and stained by fluorosis. **Non Vital Teeth:** in association with internal tooth whitening.

CONTRAINDICATIONS:

Patients with hypersensitivity to the product.

STORAGE AND PRESERVATION:

Keep syringe tightly closed, at room temperature, up to 25 °C, away from light and heat. Unused syringes within the expiration date should be kept in the refrigerator.

MODE OF USE:

Office Whitening - **Non-vital teeth:** 1. With a periapical radiograph, ensure that the endodontic treatment is adequate; 2. Clean the pulp chamber with a spherical drill, removing carious tissue and/or remains of cement and old restorations; 3. Clear about 2 to 3 mm of the root canal in the cervical region by removing the filling material; 4.

Wash the pulp chamber several times with the aid of an air/water spray and then dry properly with gentle air jets and small cotton balls; 5. With a composite resin (adhesive technique) or a restorative ionomer cement, seal the area at the mouth of the canal; 6. Isolate the gingival tissue of the teeth that will receive the bleaching gel with a light-curing resin specific for this purpose (gingival barrier), preventing the gel from touching the soft tissues; 7. Then fill the pulp chamber and cover the tooth buccal and palatal sides simultaneously with the bleaching gel, keeping the gel in contact with the tooth structure for 45 minutes. After this time, remove the gel, aspirating it with the help of an aspirator. The remaining product must be removed with an air/water jet and the tooth must then be dried; 8. Then fill the pulp chamber with bleaching gel, cover the gel with a small cotton ball and seal the pulp chamber with composite resin (use an adhesive technique for this); 9. Remove the gingival barrier; 10. Check occlusion at maximum habitual intercuspation, protrusion and lateroprotrusion. If there are any interferences, with the aid of a spherical diamond tip, remove them by grinding the resin; 11. Keep the dressing on for about 1 to 3 days and then remove it. If necessary, repeat steps 6 and 7. These steps can be repeated several times until the tooth presents the desired coloration. They can even be repeated daily; 12. After the tooth is properly whitened (preferably a little more than the neighboring teeth), restore the pulp chamber using composite resin and an adhesive technique; 13. The same whitening gel can also be used in 45-minute sessions to whiten vital teeth. **Vital teeth:** 1. Perform arch prophylaxis, before starting the treatment; 2. With the aid of a color scale, note the original color of the teeth and/or obtain photos to serve as comparative documentation; 3. Isolate the gingival tissue of the teeth that will receive the bleaching gel with a light-curing resin specific for this purpose (gingival barrier), preventing the gel from touching the soft tissues; 4. Apply, both vestibular and palatal of the teeth to be bleached, the 37% carbamide peroxide (Power Bleaching 37%); 5. The product should remain in contact with the dental structure for 45 minutes and then be aspirated with the aid of an aspirator. The

remaining product must be removed with air/water jet; 6. Remove the gingival barrier; 7. In case of tooth sensitivity apply Painless 5% (desensitizing agent) for 10 minutes prior to the application of Power Bleaching. Note: The whitening sessions for vital teeth can be repeated as many times as necessary, with no minimum waiting time between sessions.

WARNINGS AND PRECAUTIONS:

It is intended exclusively for in-office application by the dentist. The dentist should wear gloves and protective glasses during the application of the product. The dentist should inform the patient of situations in which there may be partial success or failure of treatment, such as teeth that have suffered trauma, mineralization of the pulp chamber and root canal, considerable color differences between teeth, presence of restorative materials and prosthetic restorations, among others. Children: This product should be kept out of the reach of children, especially since the accompanying syringe is an object of attractive color and appearance.

ADVERSE REACTIONS:

During the whitening procedure, the patient may experience some type of dental or gingival sensitivity.