PRESENTATION:

1 Syringe with 2 g and 5 tips.

COLORS:

Yellow / Light / Clear / Yellow + / Light +.

INDICATION:

NATURES TRY-IN is a product that is used so that the professional can test the color to be used before making the final cementation. That is, a test cementing material (Try-in) glycerin-based, mineral charge substances and colorants that enable a preview of the final result.

COMPOSITION:

Glycerin, preservatives, polyethylene glycol, silicon dioxide, pigments and purified water.

MODE OF USE:

1. Remove with brush / cup the prophylaxis paste provisional facet and temporary cement residues;
2. Select the color of NATURES TRY-IN paste according to the color of facet created and the adjacent teeth;
3. Then, dry the tooth's surface and put little amount of NATURES TRY-IN paste within the piece. Put it on the tooth to be restored. Remove the excess paste;
4. Check the color combination effect created. If necessary, try other colors, proceed as described above;
5. Remove the workpiece and carefully wash repeatedly the piece and the tooth surface with water.

WARNINGS AND PRECAUTIONS:

Exclusive use for professional dentists. Follow the instructions carefully. Keep out of reach of children and pets. Do not use the product past the expiration date. Do not reuse empty packaging. Avoid contact with skin. In case of contact, remove excess with hydrophilic cotton soaked in water and rinse with plenty of water repeat as necessary. Avoid eye contact. In case of contact with eyes, rinse immediately with plenty of water and seek medical assistance. Check correct syringe closure after use. In cases of allergic reactions to the product, discontinue use immediately.

CONTRAINDICATIONS:

The product should not be used by people who have known sensitivity to any component of the formula.

STORAGE:

Keep product away from direct sunlight. Temperature: 5 °C - 30 °C (41 °C - 86 °F).

EXPIRING DATE:

If properly packaged, the expiration date on the packaging represents the maximum period for using the product. Do not use after the expiration date.

DISPOSAL MODE:

Leftovers or residues of the used materials should not be reused, they should be disposed of according to local legislation. After use, dispose of the tips according to local legislation.

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