CONTENT:

1 syringe with 3 g and 1 tip.

DESCRIPTION:

Painless has the purpose of reducing or eliminating dental sensitivity caused by chemical, thermal, or osmotic stimuli. While sodium fluoride has an occlusive effect on the dentin tubules from the deposition of fluorapatite crystals, potassium oxalate has a neural action. It does not interfere the pulp homeostasis nor bleach the tooth or the mucosa. It does not burn or damage the oral tissues nor require complex instrumentation for application. It has a pleasant taste, making the application easier.

COMPOSITION:

Potassium Oxalate, sodium fluoride, humectants, preservatives, thickener, purified water.

INDICATION:

Painless is indicated for treatment of teeth having moderate to high sensitivity, resulting from thermal variations and ingestion of acid food, as well as resulting from periodontal and tooth whitening treatment.

CONTRAINDICATIONS:

Patients with hypersensitivity to the product.

INSTRUCTIONS FOR USE:

Clinic - Painless 5%: The product must be used by indication of the dentist only. After the prophylaxis, the desensitizing gel must be applied evenly on the teeth presenting sensitivity, by means of the application needle. The product must remain in contact with the dental structure by 10 minutes. The desensitizing gel must then be removed with cotton and plenty of water. The number of applications depends on the need for each case.

Home - Painless 1.5%: Provide a teeth tray similarly to those used in teeth whitening. If the patient already has a teeth whitening tray, they can use it. Apply Painless inside the teeth tray onto the spaces regarding the teeth, put it into the mouth and use for 10 minutes. The period and the number of times for each application may vary

because of the level of sensitivity. After removal of the teeth tray, the patient must brush their teeth and remove the gel from inside the teeth tray. After this procedure, the patient must dry and keep it inside the tray case.

WARNINGS AND PRECAUTIONS:

Clinic - Painless 5%: In case of contact with the skin, eyes, or oral cavity soft tissues, rinse with plenty of water. For the "patient", it is recommended, while Painless is applied, to isolate the mucosa and to control the ingestion with the suction tube. The air-water jet applied after 10 minutes to remove the desensitizing gel serve also as a test to verify the quality of the treatment, which must not cause sensitivity.

Home - Painless 1.5%: In case of any hypersensitivity to the product, the patient must suspend the use and consult the dentist immediately. The patient must take care not to swallow the desensitizing gel.

CHILDREN: This product must be kept away from children, especially because the syringe included is an attractive object.

ADVERSE REACTIONS:

During the application of the desensitizing gel, the patient may have some gingival sensitivity.

STORAGE CONDITIONS:

Keep the syringe tightly closed, at room temperature up to 25 °C, away from sunlight and heat.

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